


TrabEx PRO™

Set Up & Technique

PHACO SYSTEM SETUP REFERENCE GUIDE

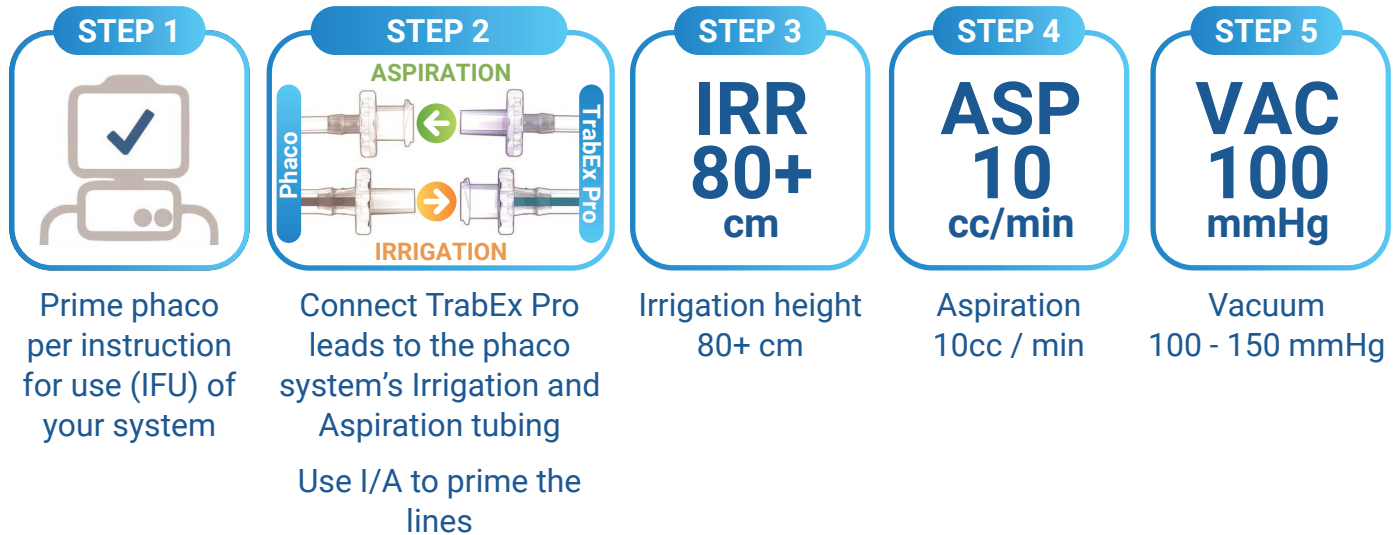

SURGICAL STEPS

Gently apply outward pressure, compressing the TM against the outer wall, creating a wrinkle in the TM. Pierce the TM with the blunt tip so it enters Schlemm's Canal. Relax the outward pressure allowing for a smooth surgical sweep. Advance the tip of the Handpiece along Schlemm's Canal in the direction you intend to cut TM until the desired endpoint is reached. Rotate the blade in the center of the eye and repeat the TM excision in the opposing direction.

